
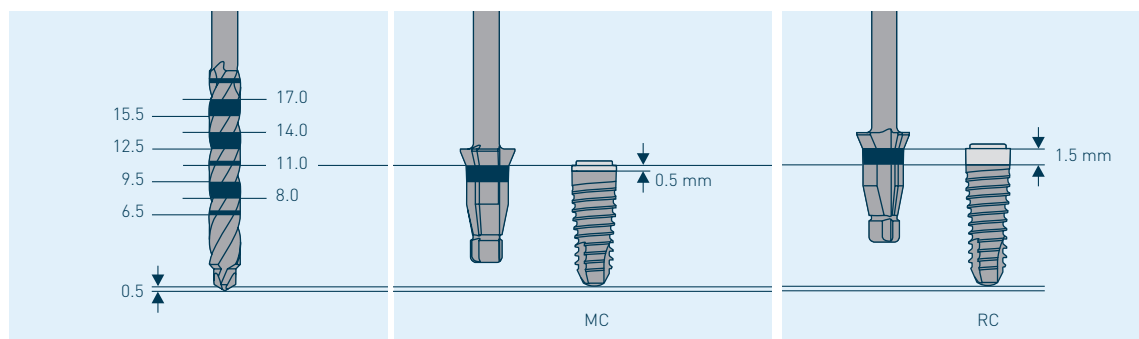
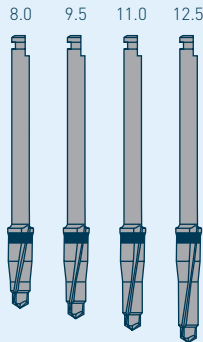


DRILLING PROTOCOL FOR **CONTACT**

PF Ø 3.5 endosseous Ø 2.7	PF Ø 4.0 endosseous Ø 3.5	PF Ø 4.5 endosseous Ø 3.5	PF Ø 5.0 endosseous Ø 4.2	PF Ø 6.0 endosseous Ø 5.0	Rotational speed rpm
2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	800
	2.8 VECTOdill	2.8 VECTOdill	2.8 VECTOdill	2.8 VECTOdill	600
			3.5 VECTOdill	3.5 VECTOdill	500
				4.3 VECTOdill	400
2.7/3.5 CONTACT Profile drill	3.5/4.0 CONTACT Profile drill	3.5/4.5 CONTACT Profile drill	4.2/5.0 CONTACT Profile drill	5.0/6.0 CONTACT Profile drill	250-300

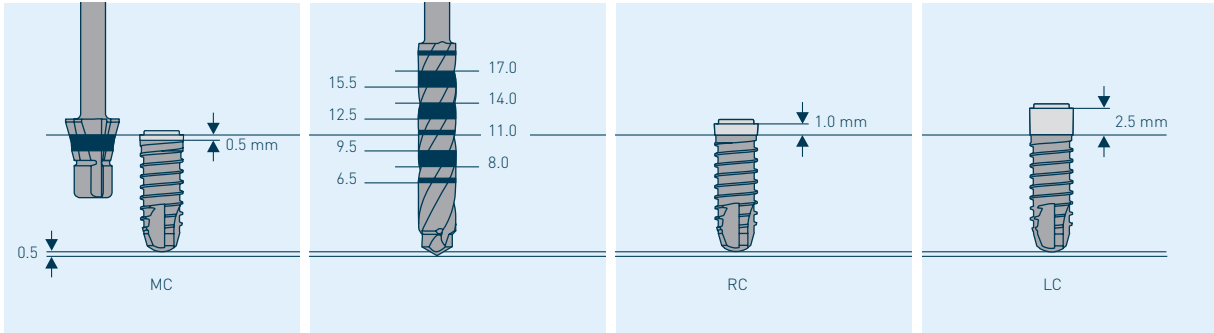

DRILLING PROTOCOL FOR **ELEMENT**


PF 3.0 endosseous Ø 2.5	PF Ø 3.5 endosseous Ø 3.5	PF Ø 4.0 endosseous Ø 4.0	PF Ø 4.5 endosseous Ø 4.2	PF Ø 5.0 endosseous Ø 5.0	PF Ø 6.0 endosseous Ø 6.0	Rotational speed rpm
2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	2.0 Pilot drill	800
 Length-specific profile drill	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	600
		3.5 VECTOdrill	3.5 VECTOdrill	3.5 VECTOdrill	3.5 VECTOdrill	500
				4.3 VECTOdrill	4.3 VECTOdrill	400
 8.0 9.5 11.0 12.5					5.3 VECTOdrill	300
	3.5/3.5 ELEMENT Profile drill *	4.0/4.0 ELEMENT Profile drill *	4.2/4.5 ELEMENT Profile drill *	5.0/5.0 ELEMENT Profile drill *	6.0/6.0 ELEMENT Profile drill *	250-300
						

* only with ELEMENT MC

Overview drilling protocol **NEVO**


PF 3.5 endosseous Ø 3.5	PF 4.0 endosseous Ø 4.0	PF 4.5 endosseous Ø 4.2	PF 5.0 endosseous Ø 5.0	Rotational speed rpm
2.0 pilot drill	2.0 pilot drill	2.0 pilot drill	2.0 pilot drill	800
2.8 VECTOdriLL™	2.8 VECTOdriLL™	2.8 VECTOdriLL™	2.8 VECTOdriLL™	600
	3.5 VECTOdriLL™	3.5 VECTOdriLL™	3.5 VECTOdriLL™	500
			4.3 VECTOdriLL™	400
NEVO/ELEMENT profile drill	NEVO/ELEMENT profile drill	NEVO/ELEMENT profile drill	NEVO/ELEMENT profile drill	250-300

Scan the QR code for the
video tutorial

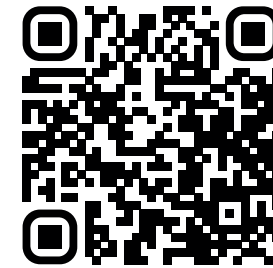


Drilling protocol for soft and medium bone


NEVO RC, PF 4.0, L 11.0 mm, supracrestal

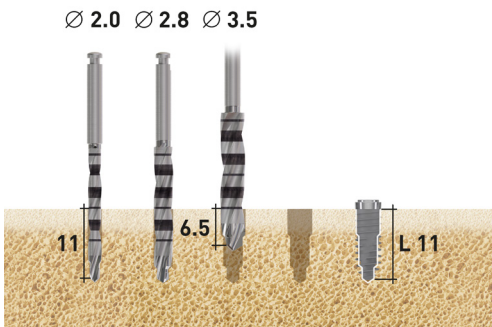


NEVO RC, PF 4.0, L 11.0 mm, subcrestal

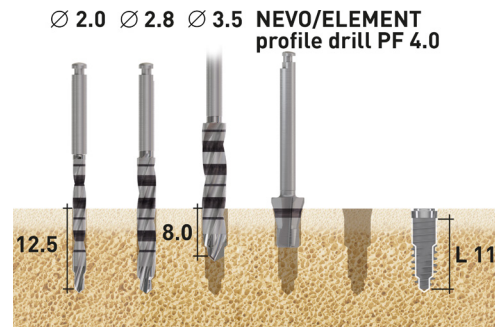

NEVO MC, PF 4.0, L 11.0 mm, epicrestal

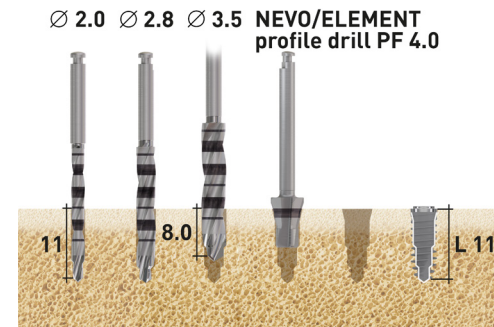


NEVO LC, PF 4.0, L 11.0 mm, supracrestal

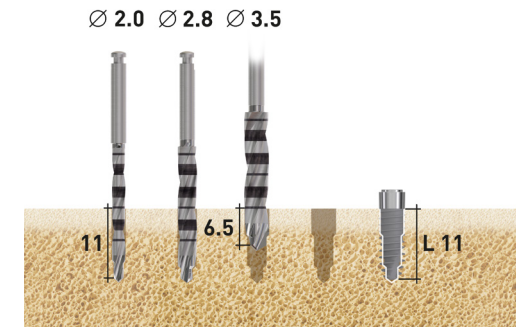


Drilling protocol for hard bone


NEVO RC, PF 4.0, L 11.0 mm, supracrestal

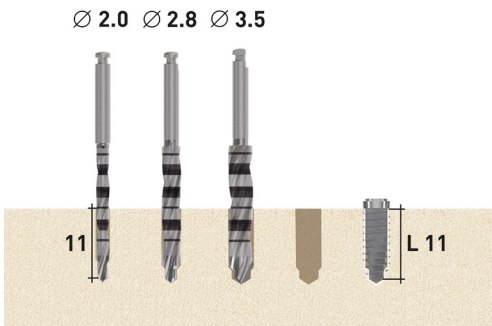

NEVO RC, PF 4.0, L 11.0 mm, subcrestal

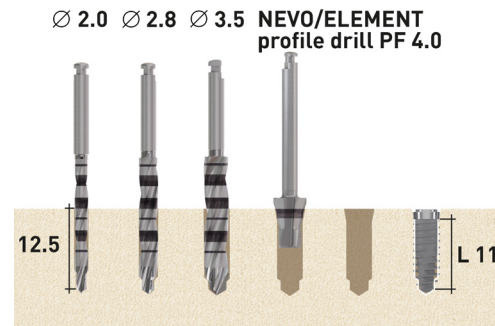

NEVO MC, PF 4.0, L 11.0 mm, epicrestal

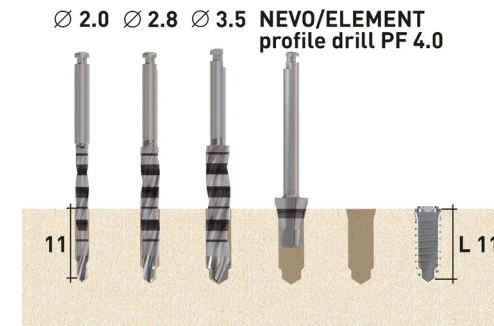

NEVO LC, PF 4.0, L 11.0 mm, supracrestal

